STYLUS

Mid-segment Aluminum Rollator

User's manual (EN)

Mittelsegment-Aluminiumrollator

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)

CE UK
CA

ENGLISH

1. General Information

Dear Customer,

This user manual contains a description of the medical device and important guidelines to ensure correct and safe use of the product. It is important to read this manual carefully prior to use. It is especially important to read the safety requirements and follow these.

REHASENSE continuously develops its products and reserves the right to change the specifications and functions of products without notice.

If you have any queries please contact your dealer or **Rehasense** directly. Contact information is located on the last page of this manual.

2. Intended usage, Indications & Contra indications

Intended usage: The rollator has been designed as an assistance walking tool for people who have mobility problems with standing, walking or seating. Thanks to the rollator those persons can stand, walk and rest easily and they can fulfil without others assistance most of their daily tasks and routines. Any other usage is prohibited.

Indications: The device is specifically indicated for individuals who, (because of a wide variety of possible health problems) require additional stable support during walking. It is specially designed for users with arthritis and/or limited strength in their arms and hands who cannot support on their wrists but still can use their forearms for support.

Contraindications: The device must not be used by persons who have serious problems with body balance or who's muscle strength is too low to be able to stand and walk, even with extra support of a rollator. The need and possibility of usage of a rollator by user should be always estimated and indicated by a physician or physical therapist.

3. Delivery & general product description

The rollators are supplied in boxes and come folded, for easier transportation.

The contents are:

- ✓ 1 complete, folded rollator included backrest and shopping bag and crutch holder
- ✓ 1 user's manual

The **STYLUS** is a four wheel, light weight rollator with a cross foldable frame. The product is made of aluminum, so it is light and easy to store or transport. The rollator offers great maneuverability, stability and its push handles are height adjustable for a good user fit. The product consists of a powder coated foldable structure frame made of aluminum extruded tubing and a foldable cross. It is also equipped with a fabric seat and a shopping bag which allows the user to rest and to carry small items.

A set of delivered additional equipment depends on order specification.

4. Security requirements

Make sure that this user's manual is read by all people who use the device. The manufacturer does not take any responsibility for damages and/or injuries caused by the fact that the user's manual has not been followed.

Limitations for usage

- The device must not be loaded with more than 150 kg.
- The device's bag has a load limitation of 5 kg.
- The device is devised to be used indoor and/or outdoor on flat surfaces. The device is not intended to be self-propelled while seated
- The brakes **MUST** be in the locked position **BEFORE** using the seat.

Avoiding accidents


- Use the device only for the purpose it is intended for.
- Use the device only in good technical condition.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device, the user must take care not to trap their fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the rollator is properly balanced and should avoid accidents.
- When using the rollator in a stationary position the hand brakes **MUST** be locked.

IF ANY SERIOUS INCIDENT OCCURES IN RELATION TO THE DEVICE PLEASE CONTACT YOUR DEALER OR REHASENSE DIRECTLY (contact details at the end of the user manual). The immediate actions should be taken according to the requirements of Regulation of the European Parliament and of the EU Council 2017/745 on medical devices of April 5, 2017.

Warning: Fingers or other body parts of a user and/or assistant can be trapped between the device's parts during folding, unfolding or adjusting of the device. Be extremely careful when adjusting the device. Observe the dimensions of gaps between the device's elements to avoid trapping your fingers or other body parts.

5. Product details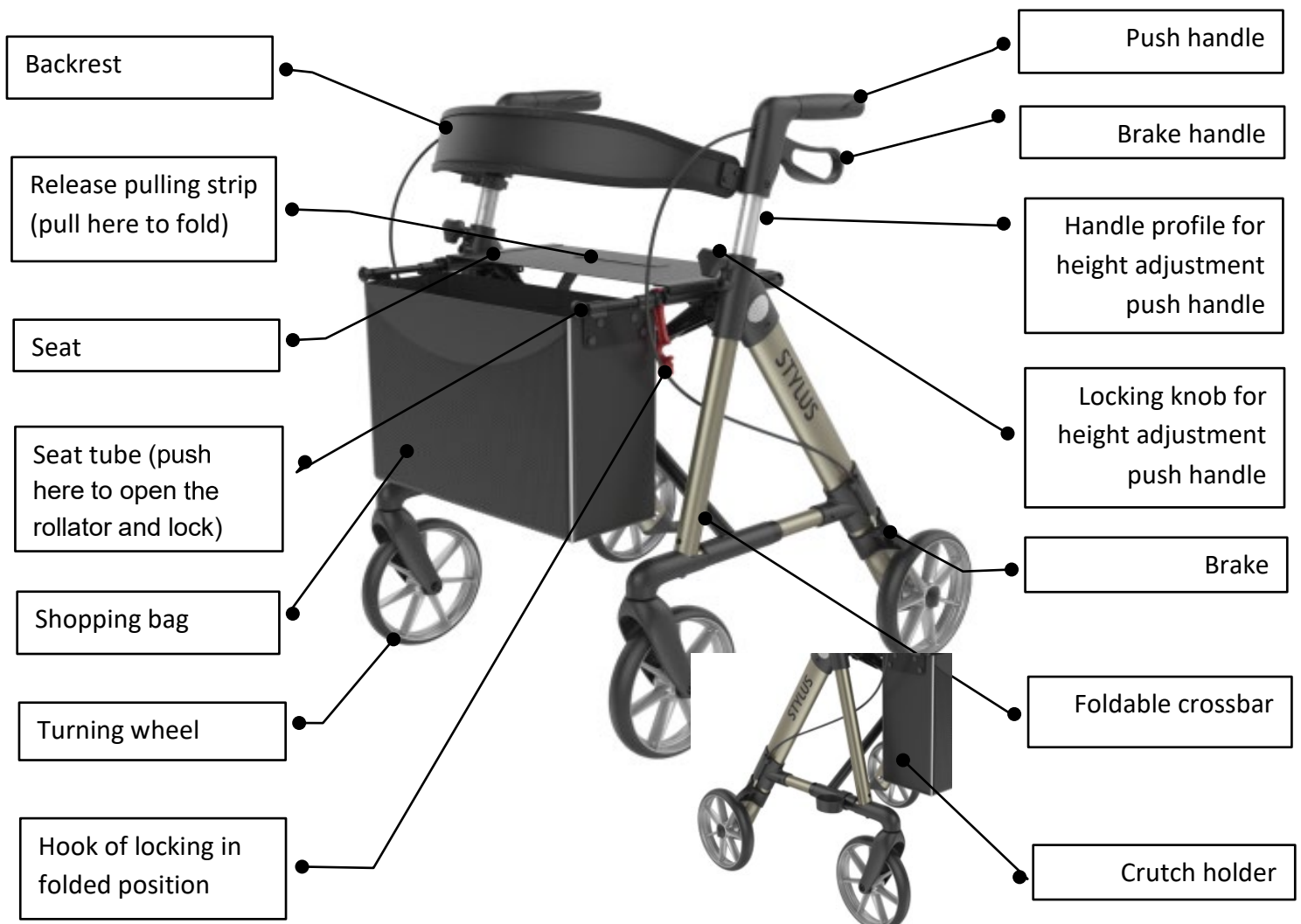

Above photo shows an example of the product with all standard components and their position. The exact appearance of your rollator and its features may differ from those shown above (depending on order specification), but their names, functions and locations remain the same.

6. Usage

Looking at the picture above the device should be prepared for use in an instinctive way. However we recommend you follow the steps below:

Initial Check

- In the first instance take the device out of the box and check (using the picture above) that nothing is missing. If you discover at this stage that some part is missing or damaged do not continue to assemble the product. Contact your supplier of the product right away.

Opening of the rollator

- Open the red hook which is locking the seating tubes
- Hold both seat tubes simultaneously push them down until you hear a “click” sound of the locking mechanism of the frame.
- Try to pull the seat tubes back up to fold the frame. If this is impossible to do the frame is opened and secured correctly. See the OPEN position in the picture above

- *After unfolding or assembling the rollator always ensure that the seat support is securely locked by the pulling strip mechanism for securing the lock and that the frame does not move anymore.*
- *Always ensure that both push handle tubes are securely locked in the main support tube with the two wing knobs to prevent the accidental sliding in of the push handle tubes when forced.*
- *Always test to see that accessories are properly and safely locked in place BEFORE using.*
- *Failure to follow these warnings can result in personal injury.*


Folding

- Pull the handle on the seat upwards so that the rollator starts to fold.
- Fold the rollator further with the tubes of the seat coming up until they touch.
- Then secure them with the red hook

Adjusting the height of the

- Release the handle locking
- Make adjustments to your
- Tighten the knob.

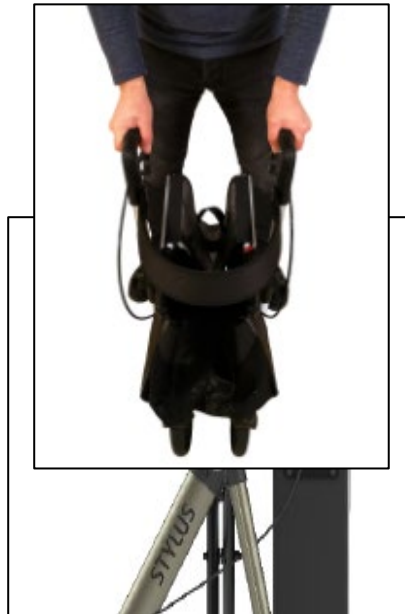push handles

knob,
desired height.

- Both push handles should be adjusted to the same height.
- The correctly adjusted handles should be to the height of the user's hips.
- Never leave the device without the breaks on.

Using the hand brakes

- TO STOP THE ROLLATOR simply pull the brake handles up towards the hand grips.
- TO REMAIN STATIONARY hold the brake handles in the "UP" position.
- TO START MOVING release the brake handles. They will drop down automatically and the brakes will be released.

- You should always engage the breaks when you don't plan to move the device.
- Never leave the device without the breaks engaged when there is someone sitting in it!
- Never leave the device on slopes without the breaks engaged!
- Never sit too much to the rear. Do not sit (partly) above the shopping bag.
- Always sit straight up when seated on the rollator with your forearms or hands lightly resting on the push handles.
- Never lean backwards when seated on the rollator

Locking/Unlocking the hand brakes

- To lock the brakes push the brake handles down until you can hear the "click" sound.
- To unlock the brakes pull the brake handles up until you feel them loosen.


- When using the rollator in a stationary position, the hand brakes **MUST** always be locked.

Tilt aid

Your walker is equipped with a tilt aid on both sides for obstacles such as door treads, curbs, etc.

- Pull the brakes, put your toes on the step, push it down and pull the handles towards you so that the front wheels come off the ground.
- Keep your heel on the floor to keep your balance.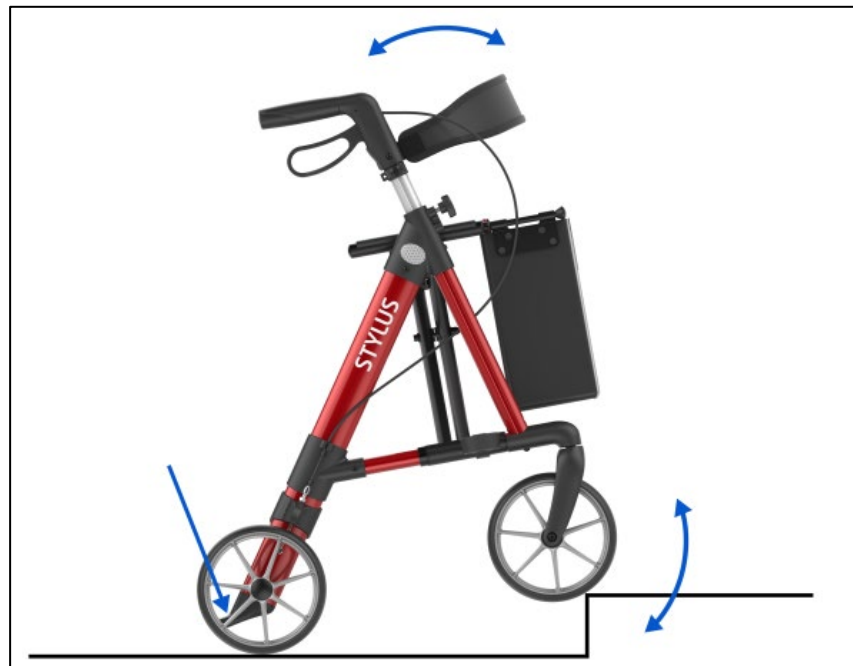

To sit on the rollator

To rest a while you can sit on the seat of the rollator.

- First put both the parking brakes on.
- Then you can turn around to take a seat.
- Sit between both handles in active seat (upright).
- You can lean on the handles with the forearms.
- Do not sit too far back.

- *Never sit too far back, not (partially) above the shopping bag.*
- *Never lean back.*
- *Never sit on the walker without the parking brake on.*


Steering and turning of the rollator

- Due to the low weight and easy swivelling front wheels, the walker is very easy to manoeuvre forwards, sideways or backwards and can be turned on the spot.
- Simply place your hand on the handles, hold the push handles and push the walker in the desired direction

Reflectors

For safety reasons, the rollator is provided with a total of 6 reflectors to be better seen in the dark: 2 are located on the side, 2 on the back of the handles and 2 reflective seams on the shopping bag.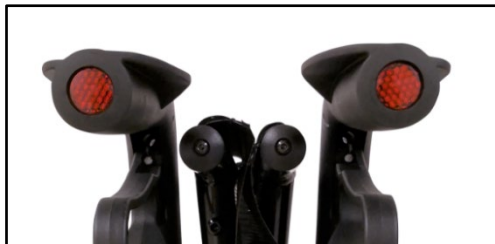

7. Cleaning & Disinfection

General:

- Clean the device regularly and keep it clean.
- Lighter dirt and dust can be removed with a wet cloth.
- Larger parts of dirt can be removed with a non-corroding, mild household detergent (available in most stores) and warm water.
- Do not use harsh abrasives or bleach based cleaners, as this may cause damage or discolouration.
- Dry parts with a soft cloth.

Wheels:

- Clean wheels with warm water and a mild cleanser mild household detergent (available in most stores). Dry with a clean rag.
- Lubricate with multi-purpose oil periodically or if wheels begin to squeak.


Disinfection:

For disinfection please contact your dealer to be sure is made by professional personnel.

However for individual use we recommend to use the generally available disinfectants without chlorine and phenol. The manufacturer is not responsible for the damage caused by wrong usage of disinfectants.

8. Label Example

The product label is placed on one of the bottom tubes of the main frame:

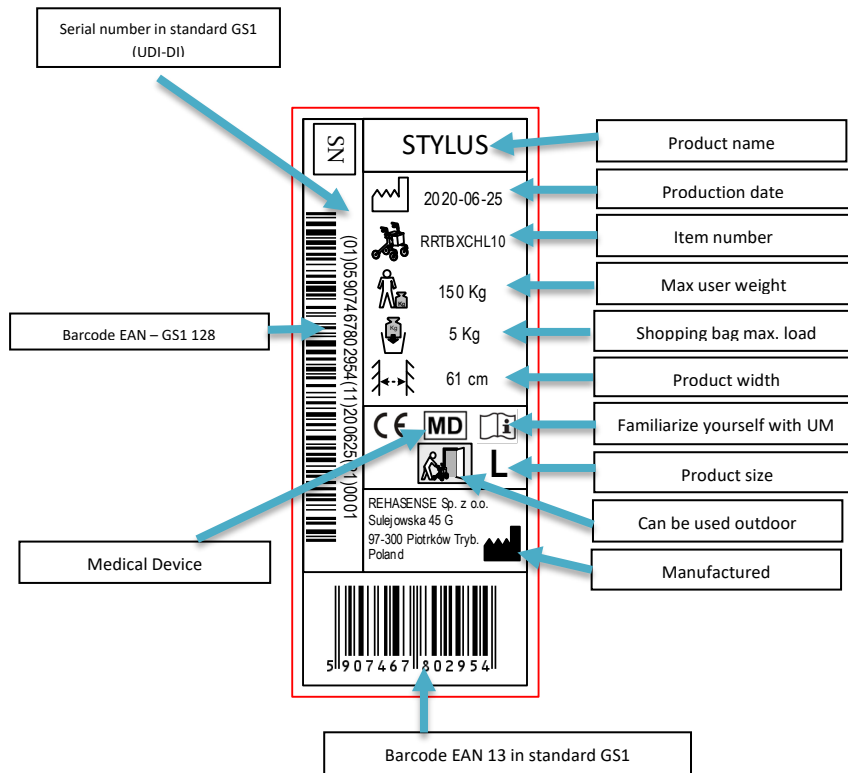9. Technical data

STYLUS – Mid-segment Aluminum Rollator	
Frame size	Large
Recommended user height	150-200 cm
Max users weight	150 kg
Weight (without accessories)	5,95 kg
Weight incl. accessories (bag, backrest and crutch holder)	6,5 kg
Height adjustable push handles	76 – 94 cm; 8 steps
The handles	Ergonomic
Distance between handles	45 cm
Length	67 cm
Width opened	60 cm
Width folded	21 cm
Height folded	81 cm
Seat width	44 cm
Seat height	60 cm
Seat depth	18,5 cm
Max. load of the bag	5 kg
Distance between handgrip and brake handle (for brake activation)	64 mm
Brakes	2 x maneuver hand brakes with parking function

Wheels (diameter x width)	200 x 35 mm
Turning radius	85 cm

10. Transport description

The rollator is an easily foldable product convenient for transportation. If it is necessary to transport it, we recommend to fold it as stated in the description above. The rollator should be transported in a way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

11. Maintenance

The rollator should be inspected.

- Ensure that every part and accessory of the rollator is secure at ALL times.
- Check all screws for tightness. If not, tighten if necessary.
- Replace any broken, damaged or worn items immediately.
- Check the braking function of both brakes and that the handbrakes perform well.

In case of any visible damages ask your local dealer for diagnostics and maintenance. We recommend to ask your dealer to carry out inspection minimum once per 12 months.

12. Re-use

This medical device can be reused by another user if the original user of the product doesn't need it anymore. Before reuse the device must be carefully inspected and disinfected by authorized technical staff at the dealer.

Following elements should be checked:

- rigidity of the structure,
- tightness of bolts and nuts,
- condition of fabric elements,
- condition of wheels and ball bearings,
- efficiency and condition of brakes.

If any of above elements are torn or broken, they should be replaced with new parts. It is specially recommended to install new wheels for every new user of the device. All lose bolts and nuts must be tight. The product must be precisely cleaned and disinfected.

WARNING! It is forbidden to re-use the rollator when the frame is bent or broken.

Every new user must receive the device with this user's manual attached.

13. How to deposit the product

The device must be disposed of in accordance with applicable local and national statutory regulations. Contact your local waste disposal agency or authorized reseller for proper disposal of packaging, metal frame components, plastic and material components.

14. Warranty

The guarantee is not valid by mis-intended use of the product.

The guarantee excludes damage caused by incorrect operation, accidents and/or through normal wear and tear.

Any modifications of product will cause loss of guarantee. Mechanical damages are not covered by this warranty.

For repairs during the guarantee period, please contact your local dealer.

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